


COOLSHIRT COOLWRAP

Rehab Wrap - Hip

Medium 1503 - 0004

Large 1503 - 0005

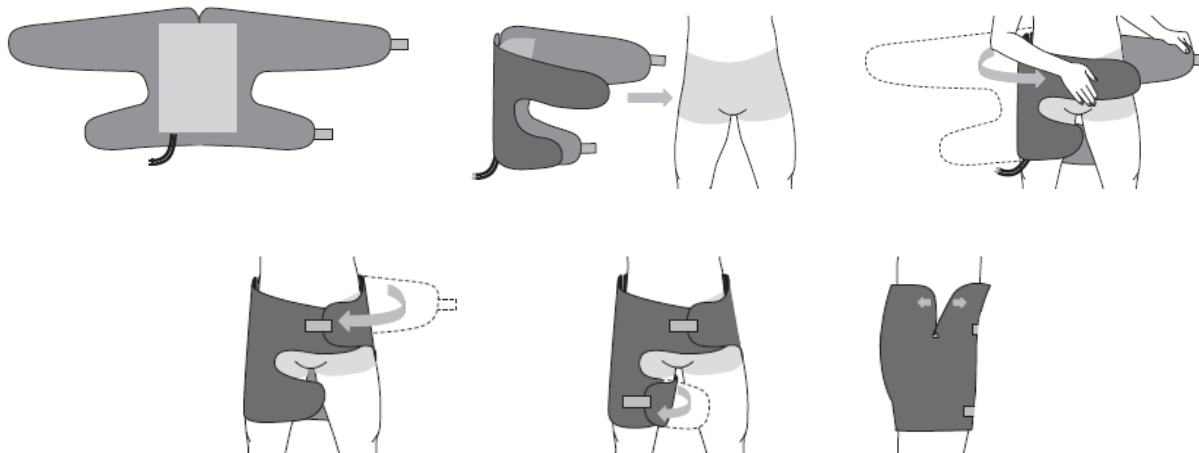

Instructions for Use

These wraps are for use only with COOLSHIRT Systems cooling engines. Do not attempt to connect to any other product or system. Read and follow the instructions for the cooling engine provided in the device user manual prior to using this product.

1. Select the appropriate size of **COOLWRAP** based on the size of the individual to be treated.
2. Place the **COOLWRAP** on the hip to be treated and secure the energy exchanger “wings” around thigh and the opposite hip using the Velcro™ tabs. Contact should be firm but not tight or constrictive.
3. Connect the **Cooling Engine** to the **COOLWRAP** hoses by pushing the quick-connectors together until firmly seated.
4. Start the desired treatment using the **Cooling Engine**.
5. Remove when the treatment is completed.

CAUTION

1. Use only under the direction of a qualified medical practitioner.
2. Never apply the energy exchanger directly to wounds or broken skin. The **COOLWRAP** is non-sterile and for single patient use. Always use a suitable sterile barrier between the **COOLWRAP** and broken skin.
3. Observe skin condition frequently due to individual differences in the sensitivity and susceptibility to injuries from cold or heat.