


## COOLSHIRT COOLWRAP

### Rehab Wrap - Multipurpose

Use for knee, lower leg, upper arm, elbow, and lower arm.

Medium            1503 - 0009  
Large              1503 - 0010

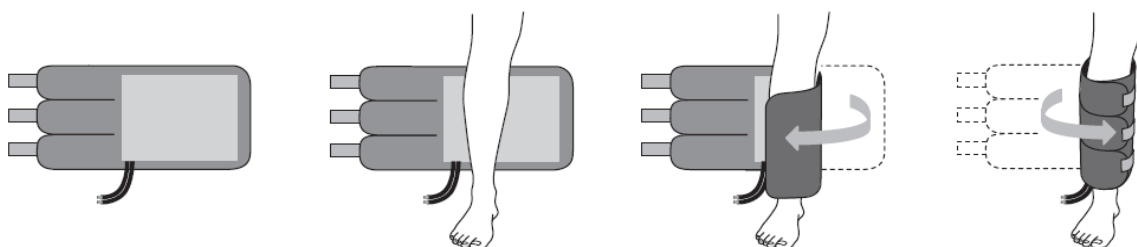

### Instructions for Use

These wraps are for use only with COOLSHIRT Systems cooling engines. Do not attempt to connect to any other product or system. Read and follow the instructions for the cooling engine provided in the device user manual prior to using this product.

1. Select the appropriate size of **COOLWRAP** based on the size of the area to be treated.
2. Wrap the **COOLWRAP** around to the area to be treated such that contact is firm but not tight or constrictive. For the knee, lower leg and upper leg, the hoses should face upward. For the upper arm, elbow and lower arm the hoses should face downward. Secure in place using the Velcro™ tabs.
3. Connect the **COOLWRAP** to the **Cooling Engine** hoses by pushing the quick-connectors together until firmly seated.
4. Start the desired treatment using the **Cooling Engine**.
5. Remove the **COOLWRAP** when the treatment is completed.

### CAUTION

1. Use only under the direction of a qualified medical practitioner.
2. Never apply the energy exchanger directly to wounds or broken skin. The **COOLWRAP** is non-sterile and for single patient use. Always use a suitable sterile barrier between the **COOLWRAP** and broken skin.
3. Observe skin condition frequently due to individual differences in the sensitivity and susceptibility to injuries from cold or heat.