


COOLSHIRT COOLWRAP

Rehab Wrap - Wrist

One Size

1503 - 0006

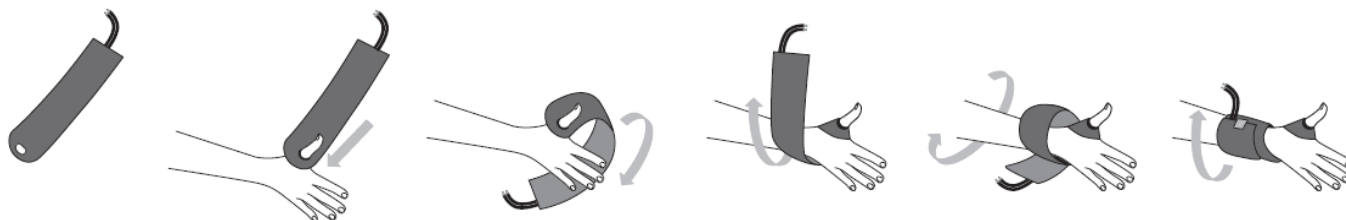

Instructions for use

These wraps are for use only with COOLSHIRT Systems cooling engines. Do not attempt to connect to any other product or system. Read and follow the instructions for the cooling engine provided in the device user manual prior to using this product.

1. Place the thumb through the hole in the **COOLWRAP** with the plastic surface facing the palm of the hand. Wrap the **COOLWRAP** around the wrist with the plastic surface against the skin and secure with the **Velcro™** tab. Contact should be firm but not tight or constrictive.
2. Connect the **COOLWRAP** to the **Cooling Engine** hoses by pushing the quick-connectors together until firmly seated.
3. Start the desired treatment using the **Cooling Engine**.
4. Remove the **COOLWRAP** when the treatment is completed.

CAUTION

1. Use only under the direction of a qualified medical practitioner.
2. Never apply the energy exchanger directly to wounds or broken skin. The **COOLWRAP** is non-sterile and for single patient use. Always use a suitable sterile barrier between the **COOLWRAP** and broken skin.
3. Observe skin condition frequently due to individual differences in the sensitivity and susceptibility to injuries from cold or heat.